



# OVERBED TABLES

## SETUP GUIDE



Thank you for choosing this Aspire Overbed Table.

This product has been designed to offer supreme stability, safety and adjustability.

**It is important that you read this setup guide carefully before using your Aspire Overbed Table.**

## TABLE OF CONTENTS

---

1. Overbed Components	4
2. Assembly Instructions	5
3. Adjusting Your Table	6
4. Assembly Of Split Top OBT	7

# 1. OVERBED COMPONENTS

This Overbed Table comes with the following parts:

Description	Quantity
Table Top with bracket mounted on bottom surface	x 1
Upright Post with quick adjustment lever and fixing knob	x 1
C-Shape Base Frame with 4 x black castor wheels (2 x locking)	x 1
14mm Bolts (Long) with washers and spacers (FOR BASE)	x 2
12mm Bolts (Short) with washers (FOR TABLE TOP)	x 2

### Laminate Table Top

accommodates food trays, personal items, and provides a stable writing surface

### Quick Adjustment Lever

makes height adjustment easy for users and carers

### Flat Pack Carton Design

is utilised to improve shipping / storage efficiency and reduce potential for damage

### Fixing Knob

to tighten upright in place

### Two Locking Castors

allow the table to move easily and be positioned securely



Laminate AC  
White Top  
BEA006700



Laminate  
Beech Top  
BEA006701



Thermoform Recessed  
Top (Cream)  
BEA028000



Split Tilt Top  
(Cream)  
BEA028110

# 2. ASSEMBLY INSTRUCTIONS


Open the carton carefully. DO NOT USE a box cutter or sharp implement as this may damage the product. For safety, lay the carton on its side with the top of the carton open and slide out the Overbed Table contents.



Insert spacers, 14mm bolts and washers into the mounting plate on the C-shape base (from below).



Locate 14mm bolts into the base of the upright post and tighten. Ensure the upright post is straight on the base plate whilst tightening. Loctite or other fixing agents may be used if required.

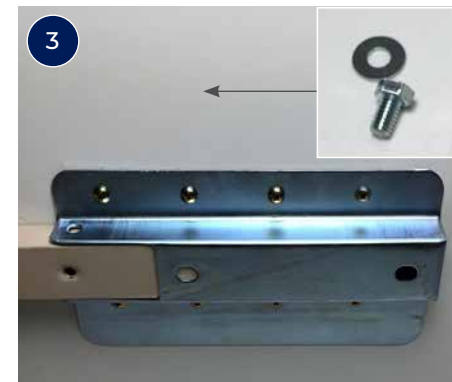

Insert upright post table support bar into table top bracket and attach using 12mm short bolts and small washers.



Remove the transportation screw from the upright and your over bed table is ready for use.

### 3. ADJUSTING YOUR TABLE

To adjust your Overbed Table:



Loosen the fixing knob on the upright post.



With one hand pull and release the quick adjustment lever and move the table up or down until the desired height is reached. Minimal force should be required.



Re-tighten fixing knob to lock in place.





### 4. ASSEMBLY OF SPLIT TOP OBT (BEA028110)

#### Tools required

- 5mm Allenkey

#### Screws and washers required (supplied in the kit)

- 4 x 5/16" Pan head screws
- 4 x 5/16" Washers

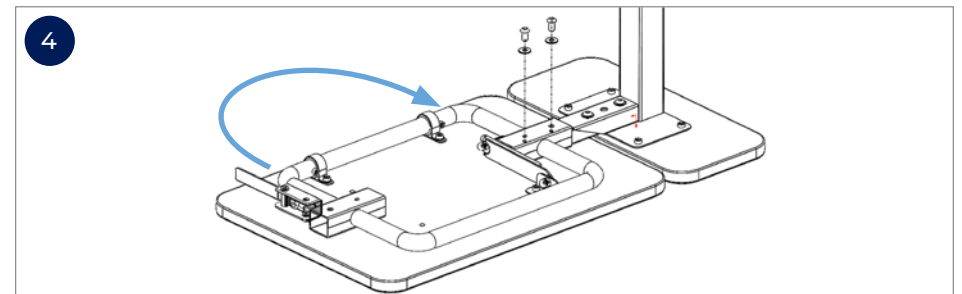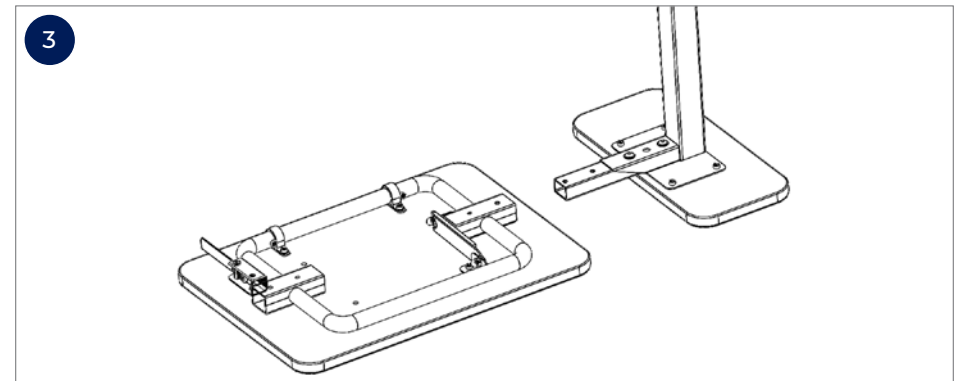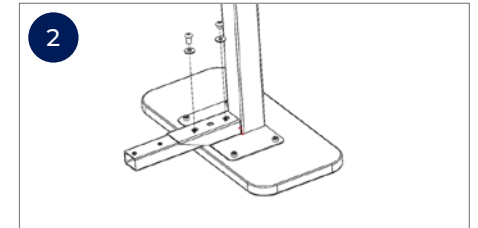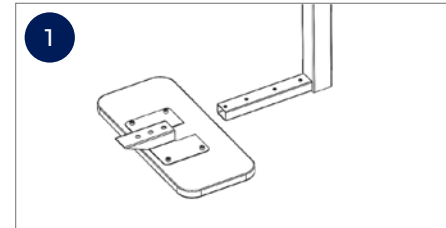

**PLEASE NOTE** table can be configured to left-hand **OR** right-hand design by rotating the split top assembly as shown with arrows **before** assembly

**AIDACARE PTY LTD**

Building 3A, 1 Moorebank Avenue,  
Moorebank NSW Australia 2170

**1300 133 120** | [aidacare.com.au](http://aidacare.com.au)

MADE IN PRC

ASUM05.10 020223

